



1 PREPARATION

Avoid the use of metal clips

15cm

ØA ØB

Diameter

2 MODEL SELECTION

Vein Graft Length	Vein Graft Diameter	
	ØA	ØB
4.0cm – 16.0cm	ØA16	ØB16
16.0cm – 23.0cm	ØA23	ØB23

3.0mm < ØA ≤ 5.5mm < ØB ≤ 8.0mm

Anastomosis / Bulldog

15cm mark

16 model

23 model

3 IMPLANTATION

Distal end Proximal end

Distal Application

Proximal Application

4 CUTTING

Hold & Cut both VEST and the Vein

5 INSPECTION

2-10mm

2-10mm